

MEDSTOCK[®]

ALGINATE DRESSING

**A HIGHLY ABSORBENT,
FAST-GELLING CALCIUM ALGINATE**

Medstock Alginate Dressing consists of an innovative blend of Mannuronic and Guluronic acid fibres with high dry and wet tensile strength.

AVAILABLE IN 4 SIZES

Key Functions and Features




- + Absorb moderate to high levels of exudate
- + Form a moist gel over wound bed for optimal wound healing
- + Alleviate pain to wound by hydrating and protecting nerve endings
- + Reduce infection risk by trapping bacteria between fibres
- + Conform to wound contours
- + Reduce foreign body retainment

General Advice on Safe Use

- + Ensure wound area is clean and dry before application
- + Can be left in place for up to 7 days depending on exudate level and type of wound, daily changes may be required
- + Alginate dressing should be trimmed to exact wound size and shape to prevent skin maceration
- + Upon removal, cleanse away all residue with saline, as any remaining fibres/gel may cause a foreign body reaction
- + Absorbent secondary dressing must be used to secure alginate dressing in place for best exudate management

Indications for Use

- + For moderate to heavy exudating wounds
- + Ulcers, sinuses and open cavities, and surgical wounds where exudate exists
- + Superficial to partial thickness burns
- + Skin donor sites

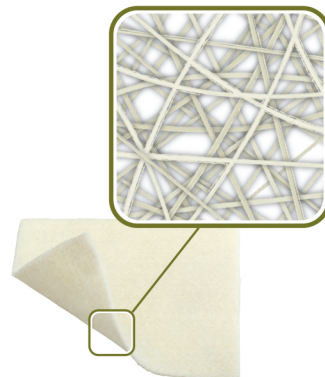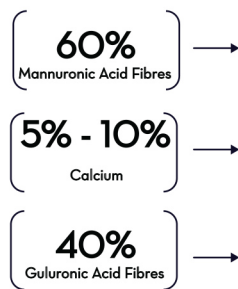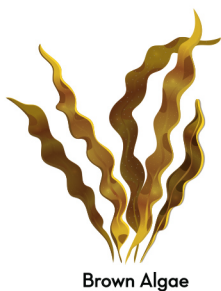

Alginate Wound Dressing



Contraindications / Precautions

- + Lightly exuding wounds may cause the dressing to adhere. If the dressing is not easily removed, use saline to moisten, then remove
- + Do not use if known allergy to calcium alginate
- + If clinical signs of infection exist, consult a physician

Storage Conditions

- + Store in a well-ventilated clean room at or below 35°C with relative humidity less than 80% and no corrosive gas

Packaging

- + Each dressing is packaged in a paper/poly closed pouch, then packed into our unique colour coded Medstock boxes

Period of Validity The product shall be valid for 5 years from the sterilisation date, provided that it is kept under the stipulated storage conditions

Care and Use Symbols



Read Instructions for use.



Single use only.



Product is latex free.



Sterilised by gamma



Keep dry.



Store out of direct sunlight.



Do not use if packaging is broken or damaged.

Product Code	Size	Pcs/Box
MS0505A	5cm x 5cm	10pcs/box
MS1010A	10cm x 10cm	10pcs/box
MS1020A	10cm x 20cm	10pcs/box
MS0230A	2cm x 20cm	10pcs/box