

**MED** **STOCK**<sup>®</sup>

**FIRST AID**

**FOAM  
NON-  
ADHESIVE  
DRESSING**

**A FLEXIBLE AND COMFORTABLE  
FOAM DRESSING THAT PROVIDES  
RAPID EXUDATE ABSORPTION**

Medstock Foam Non-Adhesive is a highly absorbent polyurethane foam dressing that contains carboxymethylcellulose (CMC) hydrophilic granules, promoting and maintaining a balanced, moist healing environment.

AVAILABLE IN 2 SIZES

## Key Functions and Features


- + High absorbency and fluid handling capacity
- + Reduces the risk of maceration to wound and periwound
- + Embedded with carboxymethylcellulose hydrophilic granules that promotes a moist wound healing environment
- + Ability to vertically wick exudate away from the wound and retain exudate in the core of the foam
- + Non-adherent to the wound preventing damage to healing tissue
- + Surface layer consists of a semi-permeable polyurethane film to protect the wound against water, bacteria and foreign matters
- + Good elasticity and flexibility which minimises the risk of pressure on the wound bed
- + Effective odour control for malodorous wounds

## Indications for Use:

- + Suitable for wounds with low to high exudate
- + Adaptable to different phases of wound healing
- + Partial thickness burns
- + Pressure ulcers
- + Venous ulcers
- + Arterial ulcers
- + Mixed aetiology ulcers

## General Advice on Safe Use

- + Ensure wound area is clean and dry before application
- + Dressing pad should overlap the wound by at least 2cm onto the dry surrounding skin
- + Can be left in place for up to 7 days but should be changed immediately if 3/4 strikethrough (visible exudate) is apparent
- + Can be used as a secondary dressing over hydrocolloid paste, alginate and other primary dressings
- + If necessary, the dressing can be cut to the required shape and size
- + Additional tape or other means of fixation must be used to secure dressing

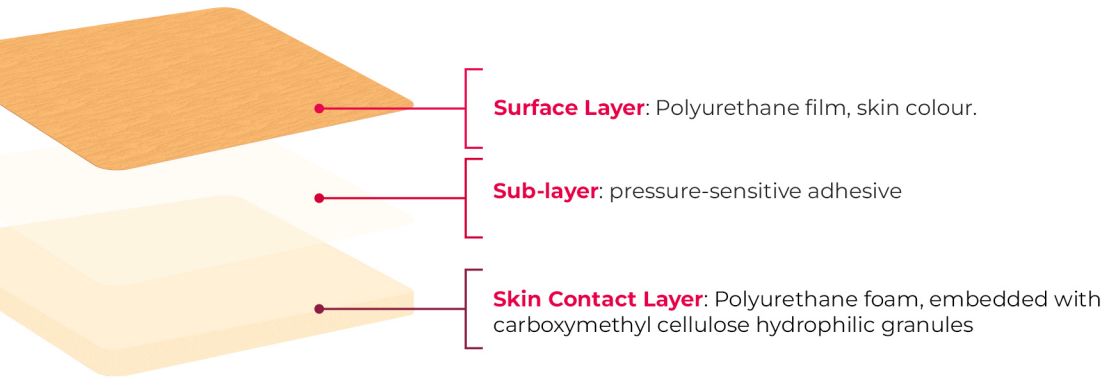

## Contraindications / Precautions

- + If reddening or sensitisation occurs, discontinue use and consult a physician

## Period of Validity

The product shall be valid for 5 years from the sterilisation date, provided that it is kept under the stipulated storage conditions

## Storage Conditions

- + Store in a well-ventilated clean room at or below 35°C with relative humidity less than 80% and no corrosive gas

## Packaging

- + Each dressing is packaged in a paper/poly closed pouch, then packed into our unique colour coded Medstock boxes

### Care and Use Symbols



Read Instructions for use.



Single use only.



Product is latex free.

STERILE R

Sterilised by radiation.



Keep dry.



Store out of direct sunlight.



Do not use if packaging is broken or damaged.

Product Code	Size	Pcs/Box
PHA0505F	5cm x 5cm	2pcs/box
PHA1010F	10cm x 10cm	2pcs/box