

MED **STOCK**[®]

FOAM

TRACHE NON
ADHESIVE

**RAPID ABSORPTION FOR
OPTIMAL BODY FLUID AND
EXUDATE MANAGEMENT**

Medstock Non-Adhesive Foam Trache Dressing is a highly absorbent polyurethane foam dressing featuring a versatile fenestration, allowing it to fit neatly around a tracheostomy tube and various other medical devices.

AVAILABLE IN 1 SIZE

Key Functions and Features


- + High absorbency and fluid handling capacity, minimising dressing changes
- + Reduces the risk of maceration to tracheal stoma sites and peristomal skin
- + Embedded with carboxymethylcellulose hydrophilic granules that promote a moist wound healing environment
- + Ability to vertically wick exudate away from the wound and hold exudate in the core of the foam
- + Non-adherent to the wound, preventing damage to healing tissue
- + Highly breathable polyurethane film on the outer surface that prevents leakage of secretions and acts as a barrier to water, bacteria, and foreign matter
- + Good elasticity and flexibility
- + Provides cushioning under medical devices to help reduce risk of pressure injuries
- + Provides odour control

Indications for Use:

- + Tracheostomy stoma sites
- + Cannula sites
- + External bone fixators
- + Various appropriately sized tubes for wound drainage

General Advice on Safe Use

- + Ensure that the wound area is clean, dry, and free from any oily residue before application
- + The dressing may remain in place for up to 7 days; however, replace it immediately if 3/4 strikethrough (visible exudate) becomes evident
- + Cut the dressing to the required shape and size if necessary
- + Alternative fixation methods must be used to secure the dressing in place

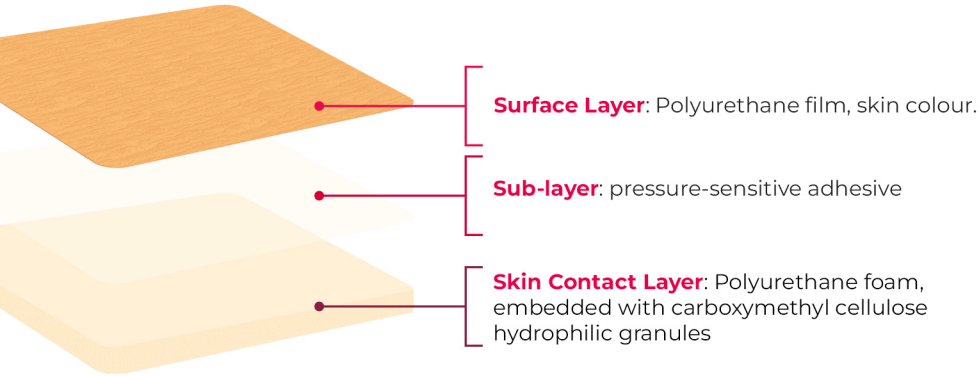

Contraindications / Precautions

- + If reddening or sensitisation occurs, discontinue use and consult a physician

Period of Validity

The product shall be valid for 5 years from the sterilisation date, provided that it is kept under the stipulated storage conditions

Storage Conditions

- + Store in a well-ventilated clean room at or below 35°C with relative humidity less than 80% and no corrosive gas

Packaging

- + Each dressing is packaged in a paper/poly closed pouch, then packed into our unique colour coded Medstock boxes

Care and Use Symbols



Read Instructions for use.



Single use only.



Product is latex free.



Sterilised by radiation.



Keep dry.



Store out of direct sunlight.



Do not use if packaging is broken or damaged.

Product Code

MS0909F

Size

9cm x 9cm

Pcs/Box

10pcs/box