

**MED** **STOCK**<sup>®</sup>

# **SILICONE FOAM LITE**

# **DRESSING**

**THIN PROFILE FOAM DRESSING  
WITH A GENTLE SILICONE CONTACT  
LAYER AND BORDER**

Medstock Silicone Foam Lite with Border is a thin profiled, 4-layer foam dressing that is gentle and conformable to fragile wound tissue.

Its perforated silicone skin contact layer provides gentle adhesion to surrounding tissue without sticking to the wound bed.

AVAILABLE IN 5 SIZES

## Key Functions and Features


- + Thin profile design provides comfortable, cushioned healing
- + Silicone contact layer seals the wound edges reducing risk of maceration
- + Soft, elastic and comfortable on the skin
- + Protective barrier against water and bacteria
- + Gentle and secure adherence
- + Atraumatic on dressing removal
- + Maintains a moist wound environment to promote healing

## Indications for Use

- + Suitable for wounds with nil to low exudate
- + Traumatic wounds (including abrasions, skin tears and blisters)
- + Surgical wounds
- + Low exuding ulcers
- + Ideal for difficult to dress areas like the feet, toes, hands and fingers
- + Protection of compromised or fragile skin

## General Advice on Safe Use

- + Ensure wound area is clean and dry before application
- + Dressing pad should overlap the wound by at least 2cm onto the dry surrounding skin
- + Can be left in place for up to 7 days but should be changed immediately if 3/4 strikethrough (visible exudate) is apparent

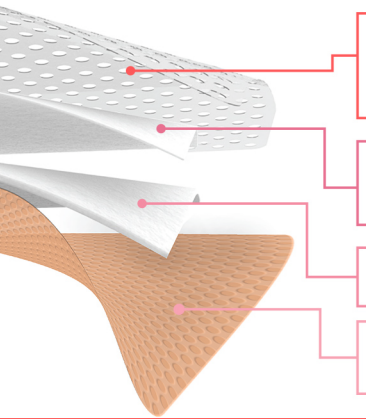

- Skin Contact Layer:** Perforated silicone gel adhesive acts to seal and secure wound edges, minimising the risk of peri-wound maceration
- Wicking Foam Layer:** Polyurethane, wicking foam which initiates vertical absorption of wound exudate away from the wound bed
- Third Layer:** Non-woven fabric layer
- Surface Layer:** Highly breathable, complexion-coloured polyurethane film that acts as a waterproof barrier to bacteria



## Contraindications / Precautions

- + Arterial bleeds and heavily bleeding wounds
- + Do not use Silicone Foam Lite with Border with oxidising agents such as hypochlorite solutions (e.g. EUSOL) or hydrogen peroxide
- + Do not cut Medstock Silicone Foam Lite with Border
- + If reddening or sensitisation occurs, discontinue use and consult a physician

## Storage Conditions

- + Store in a well-ventilated clean room at or below 35°C with relative humidity less than 80% and no corrosive gas

## Packaging

- + Each dressing is packaged in a paper/poly closed pouch, then packed into our unique colour coded Medstock boxes

## Period of Validity

The product shall be valid for 5 years from the sterilisation date, provided that it is kept under the stipulated storage conditions

### Care and Use Symbols



Read Instructions for use.



Single use only.



Product is latex free.



Sterilised using ethylene oxide.



Keep dry.



Store out of direct sunlight.



Do not use if packaging is broken or damaged.

Product Code	Size	Pcs/Box
MS0505LIT	5cm x 5cm	10pcs/box
MS0512LIT	5cm x 12.5cm	10pcs/box
MS7575LIT	7.5cm x 7.5cm	10pcs/box
MS1010LIT	10cm x 10cm	10pcs/box
MS1515LIT	15cm x 15cm	5pcs/box