

MEDSTOCK[®]

FIRST AID

SILICONE FOAM LITE DRESSING

**A THIN-PROFILE FOAM DRESSING
WITH A GENTLE SILICONE CONTACT
LAYER AND BORDER**

Medstock Silicone Foam Lite with Border is a thin-profiled, 4-layer foam dressing that is gentle and conformable to fragile wound tissue. Its perforated silicone skin contact layer provides gentle adhesion to surrounding tissue without sticking to the wound bed.

AVAILABLE IN 4 SIZES

Key Functions and Features


- + Thin profile design provides comfortable, cushioned healing
- + Silicone contact layer seals the wound edges reducing risk of maceration
- + Soft, elastic and comfortable on the skin
- + Protective barrier against water and bacteria
- + Gentle and secure adherence
- + Atraumatic on dressing removal
- + Maintains a moist wound environment to promote healing

Indications for Use

- + Suitable for wounds with nil to low exudate
- + Traumatic wounds (including abrasions, skin tears and blisters)
- + Surgical wounds
- + Low exuding ulcers
- + Ideal for difficult to dress areas like the feet, toes, hands and fingers
- + Protection of compromised or fragile skin

General Advice on Safe Use

- + Ensure wound area is clean and dry before application
- + Dressing pad should overlap the wound by at least 2cm onto the dry surrounding skin
- + Can be left in place for up to 7 days but should be changed immediately if 3/4 strikethrough (visible exudate) is apparent

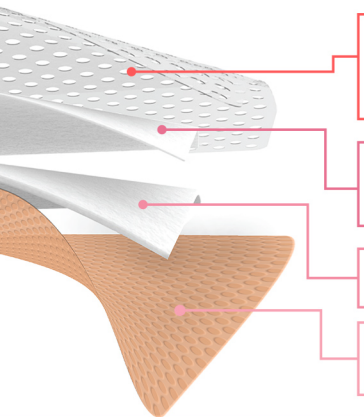

Skin Contact Layer: Perforated silicone gel adhesive acts to seal and secure wound edges, minimising the risk of peri-wound maceration

Wicking Foam Layer: Polyurethane, wicking foam which initiates vertical absorption of wound exudate away from the wound bed

Third Layer: Non-woven fabric layer

Surface Layer: Highly breathable, complexion-coloured polyurethane film that acts as a waterproof barrier to bacteria



Contraindications / Precautions

- + Arterial bleeds and heavily bleeding wounds
- + Do not use Silicone Foam Lite with Border with oxidising agents such as hypochlorite solutions (e.g. EUSOL) or hydrogen peroxide
- + Do not cut Medstock Silicone Foam Lite with Border
- + If reddening or sensitisation occurs, discontinue use and consult a physician

Storage Conditions

- + Store in a well-ventilated clean room at or below 35°C with relative humidity less than 80% and no corrosive gas

Packaging

- + Each dressing is packaged in a paper/poly closed pouch, then packed into our unique colour coded Medstock boxes

Period of Validity

The product shall be valid for 5 years from the sterilisation date, provided that it is kept under the stipulated storage conditions

Care and Use Symbols



Read Instructions for use.



Single use only.



Product is latex free.



Sterilised using ethylene oxide.



Keep dry.



Store out of direct sunlight.



Do not use if packaging is broken or damaged.

Product Code	Size	Pcs/Box
PHA0505LIT	5cm x 5cm	2pcs/box
PHA0512LIT	5cm x 12.5cm	2pcs/box
PHA7575LIT	7.5cm x 7.5cm	2pcs/box
PHA1010LIT	10cm x 10cm	2pcs/box