

**MED**STOCK<sup>®</sup>

**THIN  
HYDROCOLLOID  
DRESSING**

**A MOISTURE-DONATING, GEL-FORMING  
DRESSING TO PROMOTE HEALING  
IN DRY WOUNDS**

Medstock Thin Hydrocolloid Dressing is an effective solution for managing wounds with nil to low exudate. Its elastic, pressure-sensitive adhesive design makes it suitable for use on most body parts.

## Key Functions and Features


- + Swells in the presence of exudate, forming a gel which maintains a moist wound healing environment
- + Waterproof barrier protects wound from microbial invasion
- + May reduce pain by keeping exposed nerve endings moist
- + Facilitates autolytic debridement of wound
- + Excellent elasticity which provides comfort and flexibility, even over highly mobile areas like joints
- + Gentle on removal, protecting newly formed tissue

## Indications for Use

- + Suitable for wounds with nil to low exudate
- + Preventative management and treatment of stage I and II pressure injuries
- + Abrasions and grazes
- + Small, superficial to partial thickness burns
- + Shallow ulcers where low exudate exists
- + Skin grafts, donor sites and surgical wounds
- + Suitable for necrotic wounds to rehydrate and facilitate debridement

## General Advice on Safe Use

- + Ensure wound area is clean and dry before application
- + Dressing pad should overlap the wound by at least 2cm onto the dry surrounding skin
- + Medstock Thin Hydrocolloid dressing can be neatly cut to suit application
- + Can be left in place for up to 5 days depending on exudate level and type of wound
- + Can leave a small amount of residue in wound that should be rinsed off

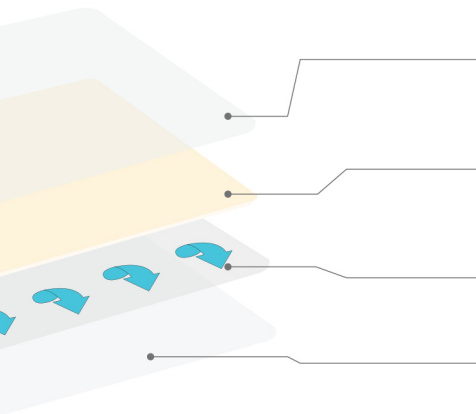

**Surface Layer:** Polyurethane film allows water vapour and oxygen to permeate through the dressing while providing a barrier to bacteria and foreign material

**Hydrocolloid Layer:** Carboxymethylcellulose granules embedded in this layer swell as exudate is absorbed, helping to maintain a moist environment conducive to wound healing

**PET Film Layer With Printing Arrow:** Transparent, plastic covering attached to the hydrocolloid adhesive and removed prior to dressing application

**Backing Release Paper:** Single-sided silicone release paper, removed after dressing application



## Contraindications / Precautions

- + Do not use on infected wounds or full thickness burns
- + Do not use if known allergy to hydrocolloid dressings
- + If reddening or sensitisation occurs, discontinue use and consult a physician

## Storage Conditions

- + Store in a well-ventilated clean room at or below 35°C with relative humidity less than 80% and no corrosive gas

## Period of Validity

The product shall be valid for 5 years from the sterilisation date, provided that it is kept under the stipulated storage conditions

## Packaging

- + Each dressing is packaged in a paper/poly closed pouch, then packed into our unique colour coded Medstock boxes

### Care and Use Symbols



Read Instructions for use.



Single use only.



Product is latex free.



Sterilised by gamma



Keep dry.



Store out of direct sunlight.



Do not use if packaging is broken or damaged.

Product Code

MS1010H

Size

10cm x 10cm

Pcs/Box

10pcs/box