

**MED** **STOCK**<sup>®</sup>

**FIRST AID**

**TRANSPARENT  
FILM**

**DRESSING**

**A HIGHLY PERMEABLE  
POLYURETHANE FILM DRESSING**

Medstock Transparent Film Dressing is a highly permeable polyurethane (PU) film dressing. It is made of a medical-grade, pressure-sensitive adhesive coating.

AVAILABLE IN 2 SIZES

## Key Functions and Features


- + Waterproof, flexible and conforms well to body contours to enhance patient comfort
- + Prevents bacterial invasion and accelerates wound healing
- + Provides protection against friction and shearing
- + Highly breathable

## Indications for Use

- + Suitable for wounds with nil to low exudate  
Ideal for epithelialising wounds
- + Waterproof film can be used to secure the fixation of a wide variety of dressings, tubes, catheters, drains and other medical devices
- + Can be used directly over superficial wounds (Superficial burns/scolds, abrasions, grazes and cuts)
- + Stage I and II pressure injuries
- + Can be used over surgical wounds to spread force and reduce suture load

## General Advice on Safe Use

- + Ensure wound area is clean and dry before application
- + Can be left in place for up to 7 days depending on exudate level and type of wound
- + Can be used as a secondary dressing to secure other primary dressings

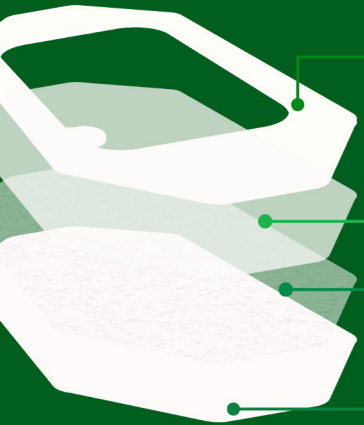

**First Layer:** CCK paper

**Second Layer:** Polyurethane (PU) film

**Third Layer:** Pressure sensitive adhesive

**Backing Release Paper:** CCK paper



## Contraindications / Precautions

- + Seek medical advice immediately for all serious wounds and burns
- + If reddening or sensitisation occurs, discontinue use and consult a physician

## Storage Conditions

- + Store in a well-ventilated clean room at or below 35°C with relative humidity less than 80% and no corrosive gas

## Packaging

- + Each dressing is packaged in a paper/poly closed pouch, then packed into our unique colour coded Medstock boxes

**Period of Validity** The product shall be valid for 5 years from the sterilisation date, provided that it is kept under the stipulated storage conditions

### Care and Use Symbols



Read Instructions for use.



Single use only.



Product is latex free.



Sterilised using ethylene oxide.



Keep dry.



Store out of direct sunlight.



Do not use if packaging is broken or damaged.

Product Code	Size	Pcs/Box
PHA0607TF	6cm x 7cm	5pcs/pack
PHA1012TF	10cm x 12cm	3pcs/pack