

MEDSTOCK[®]

TRANSPARENT FOAM ISLAND DRESSING

**A MULTI-PURPOSE FOAM DRESSING
FOR MODERATELY EXUDING
WOUNDS**

Medstock Transparent Foam Island is a transparent, highly permeable polyurethane foam dressing that incorporates carboxymethylcellulose (CMC) hydrophilic granules within the dressing to promote wound healing.

AVAILABLE IN 2 SIZES

Key Functions and Features


- + Waterproof film protects against bacterial invasion
- + Embedded hydrophilic carboxymethylcellulose (CMC) granules help to maintain a moist environment that promotes wound healing
- + Absorbs and retains exudate
- + Non-adherent, preventing damage to healing tissue
- + Gentle, flexible, and comfortable on the skin

General Advice on Safe Use

- + Ensure that the wound area is clean, dry, and free from any oily residue before application
- + The dressing foam pad should overlap the wound margins by at least 2cm onto the dry surrounding skin
- + The dressing may remain in place for up to 7 days; however, replace it immediately if 3/4 strikethrough (visible exudate) becomes evident
- + Suitable for use as a secondary dressing when applied over hydrocolloid paste, alginate, and other primary dressings

Indications for Use

- + Suitable for wounds with low to moderate exudate
- + Pressure ulcers
- + Venous ulcers
- + Arterial ulcers
- + Mixed aetiology ulcers
- + Small, partial-thickness burns
- + Surgical wounds

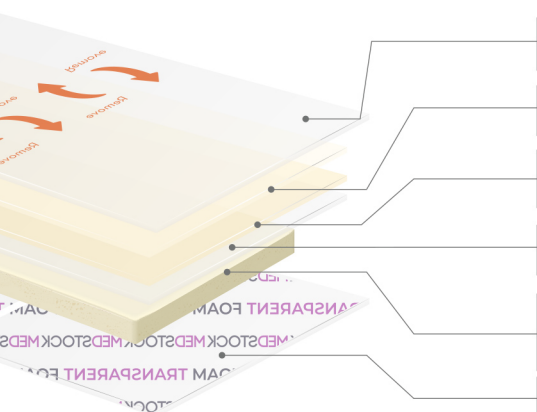

First Layer: Polyethylene Terephthalate Film (PET), printed with yellow release arrow

Second Layer: Polyethylene (PE) Film

Third Layer: Polyurethane (PU) Film

Fourth Layer: Pressure sensitive adhesive

Absorbent Layer: Polyurethane foam, embedded with carboxymethyl cellulose hydrophilic granules

Backing Release Paper: Glassine paper



Contraindications / Precautions

- + Do not use Medstock Transparent Foam Island Dressing in combination with oxidising agents like hypochlorite solutions (e.g., EUSOL) or hydrogen peroxide
- + Do not cut Medstock Transparent Foam Island Dressing
- + If reddening or sensitisation occurs, discontinue use and consult a physician

Storage Conditions

- + Store in a well-ventilated clean room at or below 35°C with relative humidity less than 80% and no corrosive gas

Packaging

- + Each dressing is packaged in a paper/poly closed pouch, then packed into our unique colour coded Medstock boxes

Period of Validity The product shall be valid for 5 years from the sterilisation date, provided that it is kept under the stipulated storage conditions

Care and Use Symbols



Read Instructions for use.



Single use only.



Product is latex free.



Sterilised using ethylene oxide.



Keep dry.



Store out of direct sunlight.



Do not use if packaging is broken or damaged.

Product Code	Size	Pcs/Box
MS7575TFI	7.5cm x 7.5cm	10pcs/box
MS1010TFI	10cm x 10cm	10pcs/box