

**MED**STOCK<sup>®</sup>

**TRANSPARENT  
ISLAND**

**DRESSING**

**HIGHLY PERMEABLE WATERPROOF  
ISLAND DRESSING**

Medstock Transparent Island Dressing is a highly permeable polyurethane (PU) film dressing coated with a quality medical-grade acrylic adhesive and featuring a gentle fabric absorption pad.

AVAILABLE IN 5 SIZES

## Key Functions and Features


- + Waterproof, conformable and comfortable to wear
- + Prevents bacterial invasion and accelerates wound healing
- + Highly breathable

## Indications for Use

- + Suitable for wounds with nil to mild exudate
- + Grazes, abrasions and superficial burns
- + Post surgical wounds

## General Advice on Safe Use

- + Ensure wound area is clean and dry before application
- + Dressing pad should overlap the wound by at least 2cm onto the dry surrounding skin  
Can be left in place for up to 5 days depending on exudate level and type of wound
- + Can be used as a secondary dressing over hydrocolloid paste, alginate and other primary dressings

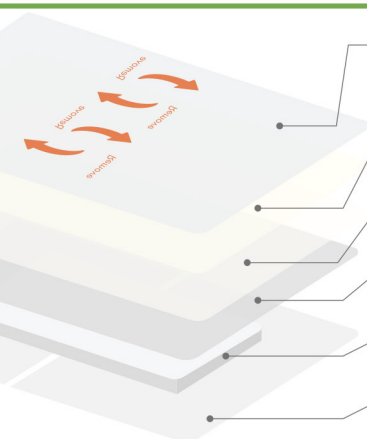

**First Layer:** Polyethylene Terephthalate Film (PET), printed with yellow release arrow

**Second Layer:** Polyethylene (PE) Film

**Third Layer:** Polyurethane (PU) Film

**Fourth Layer:** Pressure sensitive adhesive

**Fifth Layer:** Pad laminated with two-sided film

**Backing Release Paper:** Glassine paper



## Contraindications / Precautions

- + Seek medical advice immediately for all serious wounds and burns
- + If reddening or sensitisation occurs, discontinue use and consult a physician

## Storage Conditions

- + Store in a well-ventilated clean room at or below 35°C with relative humidity less than 80% and no corrosive gas

## Period of Validity

The product shall be valid for 5 years from the sterilisation date, provided that it is kept under the stipulated storage conditions

## Packaging

- + Each dressing is packaged in a paper/poly closed pouch, then packed into our unique colour coded Medstock boxes

### Care and Use Symbols



Read Instructions for use.



Single use only.



Product is latex free.



Sterilised using ethylene oxide.



Keep dry.



Store out of direct sunlight.



Do not use if packaging is broken or damaged.

Product Code	Size	Pcs/Box
MS100405TI	4cm x 5cm	100pcs/box
MS100507TI	5cm x 7cm	100pcs/box
MS100710TI	7cm x 10cm	100pcs/box
MS501012TI	10cm x 12cm	50pcs/box
MS501020TI	10cm x 20cm	50pcs/box